

# THE STANDARDIZED TENNIS ELBOW TREATMENT



- Standardised treatment method
- Compatible with all injectables
- Fully accurate injection in the tendon
- Administrable by doctor/nurse practitioner



**ITEC Medical BV offers a new patented method for treatment of a common ailment: the tennis elbow.**

The anatomically formed disposable injects through 12 micro needles the preferred injectable into the exact location of pain of the epicondylitis, the tennis elbow. The system is made with two components, the 'ITEC IT' treatment table and the 'ITEC ID' disposable. Through ultrasound, a depth measurement is performed, this depth is adjusted on the table device. The most commonly used injectables are autologous blood (untreated), dextrose, but also injection of the needles without

adding an injectable fluid.

The treatment is standardised so it can be performed with ease by a doctor or nurse practitioner.

The general practitioner now has a valid, high quality option for referral to treat this ailment.


In short time, we realised a large spread of treatment locations.



For more information, treatment locations and additional scientific studies, we would like to point you towards our website [www.itecmedical.com](http://www.itecmedical.com)

**Certification:**

CE 621544/MD 621542

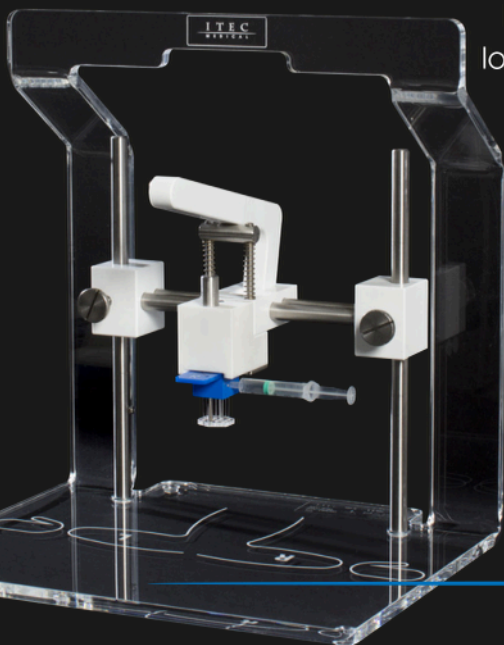

# THE STANDARDIZED TENNIS ELBOW TREATMENT

## QUICK REFERENCE CARD - ITEC MEDICAL DEVICE

ITEC  
MEDICAL




ITEC Device



marking the skin



ultrasound depth measurement

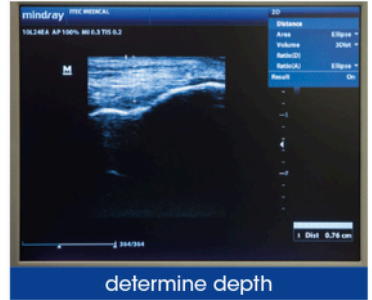

determine depth



anesthetic spray



or local anesthetic injection




depth adjustment



arm in device; shoulder 90°  
abduction, elbow 90° flexion



remove disposable from package

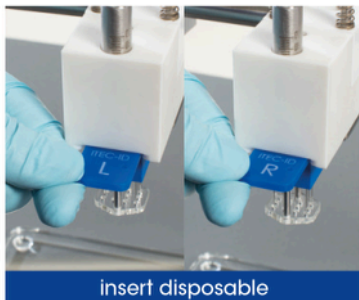

insert disposable



disinfection elbow




insert syringe



starting position: positioning  
plate adjacent to skin



remove plastic spacer

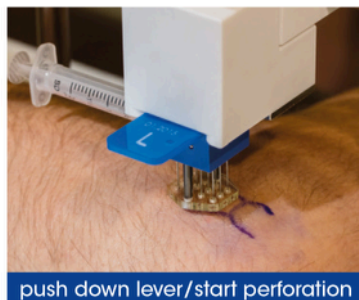

push down lever/start perforation



inject at deepest position



push up the holder



unlock the disposable



pull out the disposable



disposable in container

Version: EN\_2016\_003

ITECMEDICAL.COM



**IntegraMed**  
Healthcare Innovations

DISTRIBÜTÖR

MerdİvenkÖy Mah.YektA Sok.

Paksa Metraport A Blok

Daire : 2 GÖztepe / İSTANBUL

Tel : (0216) 355 72 37

Fax : (0216) 606 13 40

E-Posta : info@integramedikal.com

ITEC Medical B.V.

Institutenweg 25a

7521 PH Enschede

The Netherlands